



CLEANING AND DISINFECTION OF PRODUCTS MADE WITH SURE-CHEK™ FABRICS

The following disinfectants have been tested by the Herculite Products, Inc. laboratory and are safe to use on Sure-Chek fabrics when **used in accordance with the manufacturer's recommended dilution.**

Trade Name	Type	Manufacturer
For Vinyls:		
Galahad	Phenolic	Puritan
LPH	Phenolic	Steris Corp
Amphyl	Phenolic	Reckitt-Benckiser
One-Stroke	Phenolic	Steris Corp
Vesphene II	Phenolic	Steris Corp
For Urethanes:		
A33	Quaternary	Ecolab
Blue Chip	Quaternary	S.C. Johnson Co.
Elimstaph	Quaternary	Walter G. Legge
Forward DC	Quaternary	S.C. Johnson Co.
Lysol I. C. Quaternary	Quaternary	Reckitt-Benckiser
Omega	Quaternary	Ecolab
Pro-Tech RDI-36B	Quaternary	Central Solutions, Inc.
Quat Sanitizer	Quaternary	Ecolab
Sanikleen	Quaternary	West Penetone
Virex 128	Quaternary	S.C. Johnson Co.
Super SaniCloth	Quaternary/Isopropyl	Professional Disposables Inc.
SaniCloth Plus	Quaternary/Isopropyl	Professional Disposables Inc.
For Both:		
Viro-Chek	Hypochlorite	Bob Barker Co.
Alcide LD	Chlorine Oxide	Alcide Corporation

This list is not intended to be all inclusive nor should it be considered a recommendation or endorsement by Herculite Products of any product. Contact Herculite Products to question the use of a particular cleaning product on Sure-Chek fabrics or to inquire about any information offered in this bulletin.

Disinfectants: Phenolic disinfectants are the best choice for vinyl products, though properly diluted quaternaries are also acceptable. Quaternary disinfectants are recommended for urethane products. Phenolics should be avoided on urethane. Quaternary/Isopropyl disinfectants are recommended for urethanes, but not recommended for vinyls. All Sure-Chek fabrics may be cleaned with a 1:10 dilution of household bleaches containing 5.25% sodium hypochlorite as recommended by the Center for Disease Control in Atlanta, Georgia. There is no harmful effect on the fabric. Disinfectants applied at full concentration or in highly concentrated solutions will decrease the useful life of Sure-Chek fabrics. Iodophor type disinfectants used on Sure-Chek fabrics may result in staining.

Soils or Stains: Use neutral soap suds and lukewarm water. Do not use harsh cleansers, solvents or detergents.

Hard-To -Clean Spots: Use standard household/vinyl cleansers and a soft bristle brush on troublesome spots or stains. Pre-soak heavy, dried-on soil.

Laundering Vinyl-laminated or Rubber-coated Sure-Chek Fabrics is not recommended. Laundering may substantially decrease the useful life of the fabric.

Laundering of Polyurethane-coated Sure-Chek Fabrics: Machine wash with detergent up to 120°F. (Some surface wrinkling may occur. This wrinkling has no adverse effect on the fabric properties). Hang or tumble dry thoroughly at 140° F before storage.

The information contained in this bulletin is believed to be reliable. It is offered in good faith and intended for use as a general guide. Herculite Products makes no guarantee of results and assumes no obligation or liability whatsoever in connection with the use of this information. This bulletin, including any statement concerning the possible use of our products or any other manufacturer's products, is not a license to operate under or intended to suggest infringement of any existing patents.

Sure-Chek® Healthcare Fabrics

A product of

Herculite Products, Inc.

P.O. Box 435, Emigsville, PA 17318 USA

(800) 772-0036 (717) 764-1192 Fax: (717) 764-5211

email: customercare@herculite.com